


BioVendor

Research
and Diagnostic Products

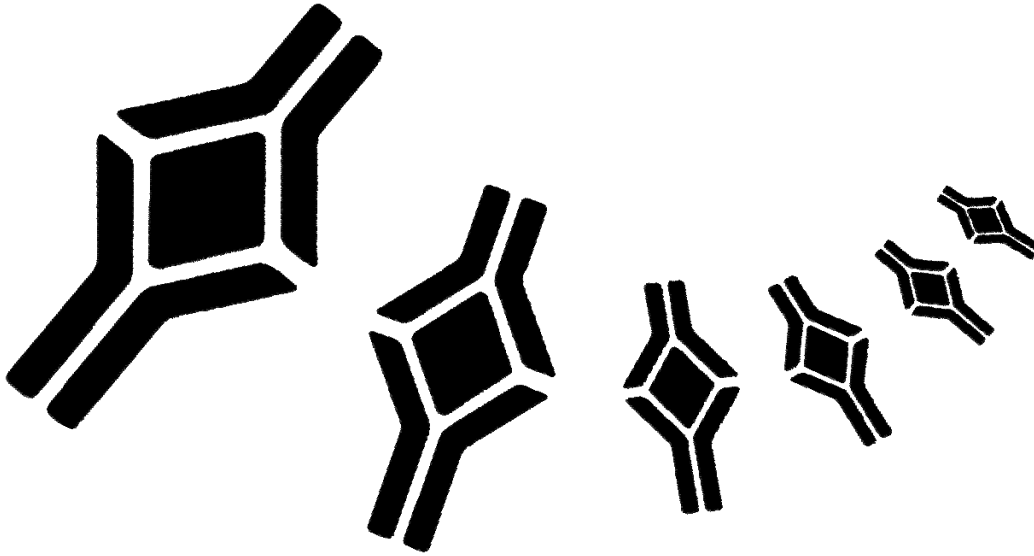

HUMAN LEPTIN RECEPTOR ELISA

Product Data Sheet

Cat. No.: RD194002100



CONTENTS

1.	INTENDED USE	3
2.	STORAGE, EXPIRATION	3
3.	INTRODUCTION	3
4.	TEST PRINCIPLE	4
5.	PRECAUTIONS	4
6.	TECHNICAL HINTS	5
7.	REAGENT SUPPLIED	5
8.	MATERIAL REQUIRED BUT NOT SUPPLIED	5
9.	PREPARATION OF REAGENTS	6
10.	PREPARATION OF SAMPLES	7
11.	ASSAY PROCEDURE	8
12.	CALCULATIONS	9
13.	PERFORMANCE CHARACTERISTICS	10
14.	DEFINITION OF THE STANDARD	15
15.	METHOD COMPARISON	15
16.	TROUBLESHOOTING AND FAQs	15
17.	REFERENCES	16
18.	EXPLANATION OF SYMBOLS	19

»» This kit is manufactured by:
BioVendor – Laboratorní medicína, a.s.

»» Use only the current version of Product Data Sheet enclosed with the kit!

1. INTENDED USE

The RD194002100 Human Leptin Receptor ELISA is a sandwich enzyme immunoassay for the quantitative measurement of human leptin receptor.

»» Features

- It is intended for in vitro diagnostic use.
- The total assay time is less than 2.5 hours.
- The kit measures leptin receptor in serum, plasma (EDTA, citrate, heparin) and tissue culture medium.
- Assay format is 96 wells.
- Quality Controls are human serum based. No animal sera are used.
- Standard is recombinant protein based.
- Components of the kit are provided ready to use, concentrated or lyophilized.

2. STORAGE, EXPIRATION

Store the complete kit at 2-8°C. Under these conditions, the kit is stable until the expiration date (see label on the box). For stability of opened reagents see Chapter 9.

3. INTRODUCTION

Leptin receptor (OB-R) was identified as a leptin binding protein (Leptin, the product of the *ob* gene, is a single-chain 16 kDa protein consisting of 146 amino acid residues.) OB-R was found to be a member of the class I cytokine receptor family with a large extracellular domain comprising 816 amino acid residues. Leptin receptor exists in multiple forms with a common extracellular domain and a variable length cytoplasmic portion. Alternate splicing from a single gene derives the six isoforms of the leptin receptor.

The soluble form of the leptin receptor, OB-R contains no intracellular motifs or transmembrane residues, thus it consists entirely of the extracellular ligand-binding domain of the receptor.

Long forms of OB-R transcripts were reported to be expressed predominantly in regions of the hypothalamus which provides evidence that leptin receptor is important in body weight regulation. Expression of short forms of OB-R transcripts have been found in multiple tissues, including the choroid plexus, lung, kidney, and primitive hematopoietic cell populations. Leptin

receptor may act as a negative regulator of leptin activity and it may maintain a pool of available bioactive leptin by binding and delaying its clearance from circulation.

Soluble leptin receptor levels are indirectly proportional to adiposity and are increased in females versus males. Leptin receptor levels are highest in infants, decrease into adolescence, and remain relatively stable throughout adulthood. Soluble leptin receptor is also found upregulated in patients with chronic heart failure, end-stage renal disease and anorexia.

4. TEST PRINCIPLE

In the BioVendor Human Leptin Receptor ELISA, Standards, Quality Controls and samples are incubated in microplate wells pre-coated with monoclonal anti-human leptin receptor antibody. After 60 minutes incubation and washing, monoclonal anti-human leptin receptor antibody, conjugated with horseradish peroxidase (HRP) is added to the wells and incubated for 60 minutes with captured leptin receptor. Following another washing step, the remaining HRP conjugate is allowed to react with the substrate solution (TMB). The reaction is stopped by addition of acidic solution and absorbance of the resulting yellow product is measured spectrophotometrically at 450 nm. The absorbance is proportional to the concentration of leptin receptor. A standard curve is constructed by plotting absorbance values against concentrations of Standards, and concentrations of unknown samples are determined using this standard curve.

5. PRECAUTIONS

- Wear gloves and laboratory coats when handling immunodiagnostic materials.
- Do not drink, eat or smoke in the areas where immunodiagnostic materials are being handled.
- This kit contains components of human origin. These materials were found non-reactive for HBsAg, HCV antibody and for HIV 1/2 antigen and antibody. However, these materials should be handled as potentially infectious, as no test can guarantee the complete absence of infectious agents.
- This kit contains components of animal origin. However, these materials should be handled as potentially infectious.
- Avoid contact with the acidic Stop Solution and Substrate Solution, which contains hydrogen peroxide and tetramethylbenzidine (TMB). Wear gloves and eye and clothing protection when handling these reagents. Stop and/or Substrate Solutions may cause skin/eyes irritation. In case of contact with the Stop Solution and the Substrate Solution wash skin/eyes thoroughly with water and seek medical attention, when necessary.
- The materials must not be pipetted by mouth.

6. TECHNICAL HINTS

- Reagents with different lot numbers should not be mixed.
- Use thoroughly clean glassware.
- Use deionized (distilled) water, stored in clean containers.
- Avoid any contamination among samples and reagents. For this purpose, disposable tips should be used for each sample and reagent.
- Substrate Solution should remain colourless until added to the plate. Keep Substrate Solution protected from light.
- Stop Solution should remain colourless until added to the plate. The colour developed in the wells will turn from blue to yellow immediately after the addition of the Stop Solution. Wells that are green in colour indicate that the Stop Solution has not mixed thoroughly with the Substrate Solution.
- Dispose of consumable materials and unused contents in accordance with applicable national regulatory requirements.

7. REAGENT SUPPLIED

<i>Kit Components</i>	<i>State</i>	<i>Quantity</i>
Antibody Coated Microtiter Strips	ready to use	96 wells
Conjugate Solution	ready to use	13 ml
Human Leptin Receptor Standards	ready to use	6 x 0.35 ml
Quality Control High	lyophilized	1 vial
Quality Control Low	lyophilized	1 vial
Dilution Buffer	ready to use	13 ml
Wash Solution Concentrate (10x)	concentrated	100 ml
Substrate Solution	ready to use	13 ml
Stop Solution	ready to use	13 ml
Product Data Sheet + Certificate of Analysis		1 pc

8. MATERIAL REQUIRED BUT NOT SUPPLIED

- Deionized (distilled) water
- Test tubes for diluting samples
- Glassware (graduated cylinder and bottle) for Wash Solution (Dilution Buffer)
- Precision pipettes to deliver 10-1000 µl with disposable tips

- Multichannel pipette to deliver 100 µl with disposable tips
- Absorbent material (e.g. paper towels) for blotting the microtiter plate after washing
- Vortex mixer
- Orbital microplate shaker capable of approximately 300 rpm
- Microplate washer (optional). [Manual washing is possible but not preferable.]
- Microplate reader with 450 ± 10 nm filter
- Software package facilitating data generation and analysis (optional)

9. PREPARATION OF REAGENTS

- All reagents need to be brought to room temperature prior to use.
- Always prepare only the appropriate quantity of reagents for your test.
- Do not use components after the expiration date marked on their label.

- Assay reagents supplied ready to use:

Antibody Coated Microtiter Strips

Stability and storage:

Return the unused strips to the provided aluminium zip-sealed bag with desiccant and seal carefully. Remaining Microtiter Strips are stable 3 month when stored at 2-8°C and protected from the moisture.

Conjugate solution

Dilution Buffer

Substrate Solution

Stop Solution

Stability and storage:

Opened reagents are stable 3 month when stored at 2-8°C.

- Assay reagents supplied concentrated or lyophilized:

Human Leptin Receptor Standards

Dilute each concentration of standard 3x with the Dilution Buffer prior to the assay, e.g. 50 µl of standard + 100 µl of Dilution Buffer for singlets, or preferably 100 µl of standard + 200 µl of Dilution Buffer for duplicates. Mix well (not to foam).

Stability and storage:

Opened standards are stable 3 month when stored at 2-8°C. Do not store the diluted (3x) standards.

Quality Controls High, Low

Reconstitute each Quality Control (High and Low) with 350 µl of distilled (deionized) water just prior to the assay. Let it dissolve at least 15 minutes with occasional gentle shaking (not to foam).

Dilute reconstituted Quality Controls 3x with Dilution Buffer, e.g. 50 µl of Quality Control + 100 µl of Dilution Buffer when assaying samples in singlets, or preferably 100 µl of Quality Control + 200 µl of Dilution Buffer for duplicates.

Stability and storage:

The reconstituted Quality Controls must be used immediately or aliquoted and frozen at -20°C for 3 months. Avoid repeated freeze/thaw cycles.

Wash Solution Concentrate (10x)

Dilute Wash Solution Concentrate (10x) ten fold in distilled water to prepare a 1x working solution. Example: 100 ml of Wash Solution Concentrate (10x)+ 900 ml of distilled water for use of all 96-wells.

Stability and storage:

The diluted Wash Solution is stable 1 month when stored at 2-8°C. Opened Wash Solution Concentrate (10x) is stable 3 months when stored at 2-8°C.

10. PREPARATION OF SAMPLES

The kit measures leptin receptor in serum or plasma and tissue culture medium.

Samples should be assayed immediately after collection or should be stored at -20°C. Mix thoroughly thawed samples just prior to the assay and avoid repeated freeze/thaw cycles, which may cause erroneous results. Avoid using hemolyzed or lipemic samples.

Dilute samples 3x with Dilution Buffer just prior to the assay (e.g. 50 µl of sample + 100 µl of Dilution Buffer for singlets, or preferably 100 µl of sample + 200 µl of Dilution Buffer for duplicates). **Mix well** (not to foam). Vortex is recommended.

Stability and storage:

Samples should be stored at -20°, or preferably at -70°C for long-term storage. Avoid repeated freeze/ thaw cycles.

Do not store the diluted samples.

See Chapter 13 for stability of serum and plasma samples if stored at 2-8°C, effect of freezing/thawing and effect of sample matrix (serum/plasma) on the concentration of leptin receptor.

Ask for protocol at info@biovendor.com if assaying other samples.

11. ASSAY PROCEDURE

1. Pipet **100 µl** of diluted Standards, Quality Controls, Dilution Buffer (=Blank) and samples, preferably in duplicates, into the appropriate wells.
2. Incubate the plate at room temperature (ca. 25°C) for **1 hour**, shaking at ca. 300 rpm on an orbital microplate shaker.
3. Wash the wells **5-times** with Wash Solution (0.35 ml per well). After final wash, invert and tap the plate strongly against paper towel.
4. Add **100 µl** of Conjugate Solution into each well.
5. Incubate the plate at room temperature (ca. 25°C) for **1 hour**, shaking at ca. 300 rpm on an orbital microplate shaker.
6. Wash the wells **5-times** with Wash Solution (0.35 ml per well). After final wash, invert and tap the plate strongly against paper towel.
7. Add **100 µl** of Substrate Solution into each well. Avoid exposing the microtiter plate to direct sunlight. Covering the plate with e.g. aluminium foil is recommended.
8. Incubate the plate for **10 minutes** at room temperature. The incubation time may be extended [up to 20 minutes] if the reaction temperature is below than 20°C. Do not shake the plate during the incubation.
9. Stop the colour development by adding **100 µl** of Stop Solution.
10. Determine the absorbance by reading the plate at 450 nm. The absorbance should be read within 5 minutes following step 9.

Note: If the microplate reader is not capable of reading absorbance greater than the absorbance of the highest standard, perform a second reading at 405 nm. A new standard curve, constructed using the values measured at 405 nm, is used to determine leptin receptor concentration of off-scale samples. The readings at 405 nm should not replace the on-scale readings at 450 nm.

Note 2: Manual washing: Aspirate wells and pipet 0.35 ml Wash Solution into each well. Aspirate wells and repeat twice. After final wash, invert and tap the plate strongly against paper towel. Make certain that Wash Solution has been removed entirely.

	strip 1 + 2	strip 3 + 4	strip 5 + 6	strip 7 + 8	strip 9 + 10	strip 11 + 12
A	Standard 100	QC Low	Sample 8	Sample 16	Sample 24	Sample 32
B	Standard 50	Sample 1	Sample 9	Sample 17	Sample 25	Sample 33
C	Standard 20	Sample 2	Sample 10	Sample 18	Sample 26	Sample 34
D	Standard 10	Sample 3	Sample 11	Sample 19	Sample 27	Sample 35
E	Standard 5	Sample 4	Sample 12	Sample 20	Sample 28	Sample 36
F	Standard 2	Sample 5	Sample 13	Sample 21	Sample 29	Sample 37
G	Blank	Sample 6	Sample 14	Sample 22	Sample 30	Sample 38
H	QC High	Sample 7	Sample 15	Sample 23	Sample 31	Sample 39

Figure 1: Example of a work sheet.

12. CALCULATIONS

Most microplate readers perform automatic calculations of analyte concentration. The standard curve is constructed by plotting the mean absorbance at 450 nm (Y) of Standards against log of the known concentration (X) of Standards, using the four-parameter algorithm. Results are reported as concentration of leptin receptor ng/ml in samples.

Alternatively, the logit log function can be used to linearize the standard curve (i.e. logit of the mean absorbance (Y) is plotted against log of the known concentration (X) of Standards.

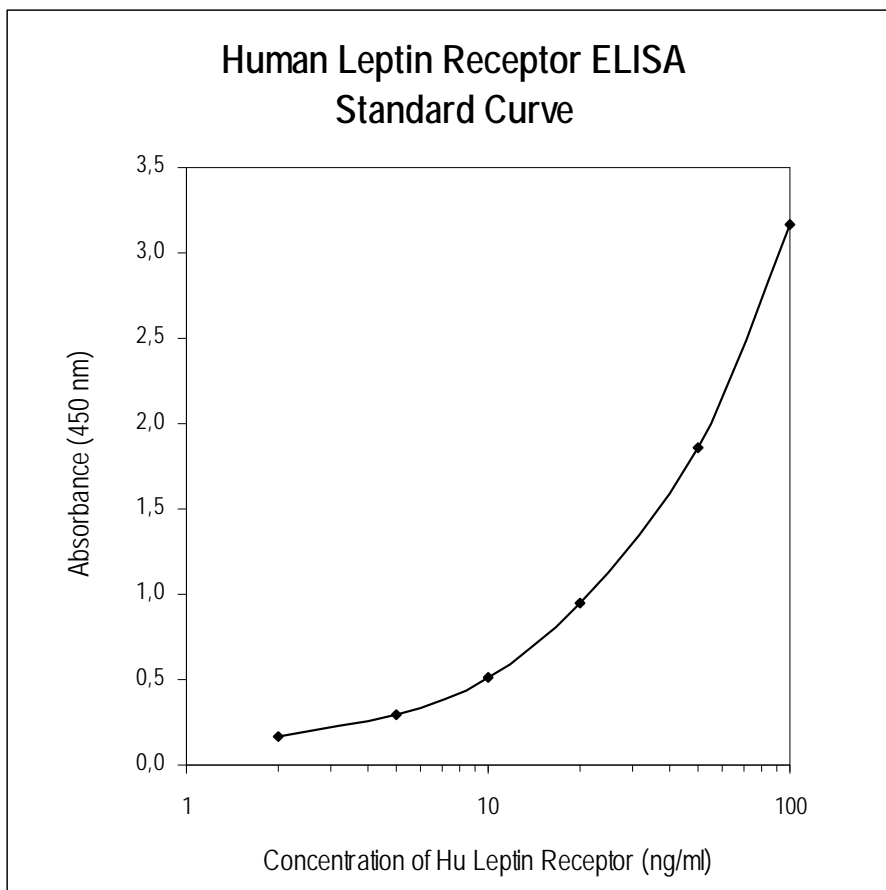


Figure 2: Typical Standard Curve for Human Leptin Receptor ELISA.

13. PERFORMANCE CHARACTERISTICS

Typical analytical data of BioVendor Human Leptin Receptor ELISA are presented in this chapter.

- **Sensitivity**

Limit of Detection (LOD) (defined as concentration of analyte giving absorbance higher than mean absorbance of blank* plus three standard deviations of the absorbance of blank: $A_{\text{blank}} + 3 \times \text{SD}_{\text{blank}}$) is calculated from the real leptin receptor values in wells and is 0.04 ng/ml.

*Dilution Buffer is pipetted into blank wells.

- **Limit of assay**

Results exceeding leptin receptor level of 100 ng/ml should be repeated with more diluted samples. Dilution factor needs to be taken into consideration in calculating the leptin receptor concentration.

- **Specificity**

The antibodies used in this ELISA are specific for human leptin receptor with no detectable crossreactivities to human cytokines.

Sera of several mammalian species were measured in the assay. See results below. For details please contact us at info@biovendor.com.

<i>Mammalian serum sample</i>	<i>Observed crossreactivity</i>
Bovine	no
Cat	no
Dog	yes
Goat	no
Horse	yes
Monkey	no
Mouse	yes
Pig	yes
Rabbit	no
Rat	no
Sheep	no

- **Precision**

Intra-assay (Within-Run) (n=8)

<i>Sample</i>	<i>Mean (ng/ml)</i>	<i>SD (ng/ml)</i>	<i>CV (%)</i>
1	23.19	1.70	7.4
2	18.12	1.17	6.5

Inter assay (Run-to-Run) (n=6)

<i>Sample</i>	<i>Mean (ng/ml)</i>	<i>SD (ng/ml)</i>	<i>CV (%)</i>
1	17.43	1.26	7.2
2	25.19	1.39	5.5

- **Spiking Recovery**

Serum samples were spiked with different amounts of human leptin receptor and assayed.

<i>Sample</i>	<i>Observed (ng/ml)</i>	<i>Expected (ng/ml)</i>	<i>Recovery O/E (%)</i>
1	13.26	-	-
	23.47	23.90	98.2
	33.33	32.48	102.6
	42.34	40.94	103.4
2	20.74	-	-
	28.57	31.38	91.0
	38.64	39.96	92.2
	45.19	48.42	93.3

- **Linearity**

Serum samples were serially diluted with Dilution Buffer and assayed.

<i>Sample</i>	<i>Dilution</i>	<i>Observed (ng/ml)</i>	<i>Expected (ng/ml)</i>	<i>Recovery O/E (%)</i>
1	-	36.10	-	-
	2x	18.45	18.05	102.2
	4x	9.38	9.03	103.9
	8x	4.64	4.51	102.8
2	-	22.83	-	-
	2x	11.44	11.42	100.2
	4x	6.31	5.71	110.6
	8x	2.90	2.85	101.6

- **Effect of sample matrix**

Citrate, heparin and EDTA plasmas were compared to respective serum samples from the same 10 individuals.

Results are shown below:

Volunteer No.	Serum (ng/ml)	Plasma (ng/ml)		
		EDTA	Citrate	Heparin
1	31.75	30.77	29.16	30.64
2	33.28	35.10	34.46	33.34
3	49.75	38.82	35.69	41.48
4	25.19	24.46	25.73	26.87
5	26.09	24.26	19.27	27.47
6	22.36	21.70	22.97	23.27
7	27.49	24.69	23.00	25.62
8	22.26	22.78	22.75	24.32
9	31.25	27.95	23.08	29.62
10	22.76	23.05	20.35	23.56
Mean (ng/ml)	29.22	27.36	25.65	28.62
Mean Plasma/Serum (%)	-	93.6	87.8	97.9

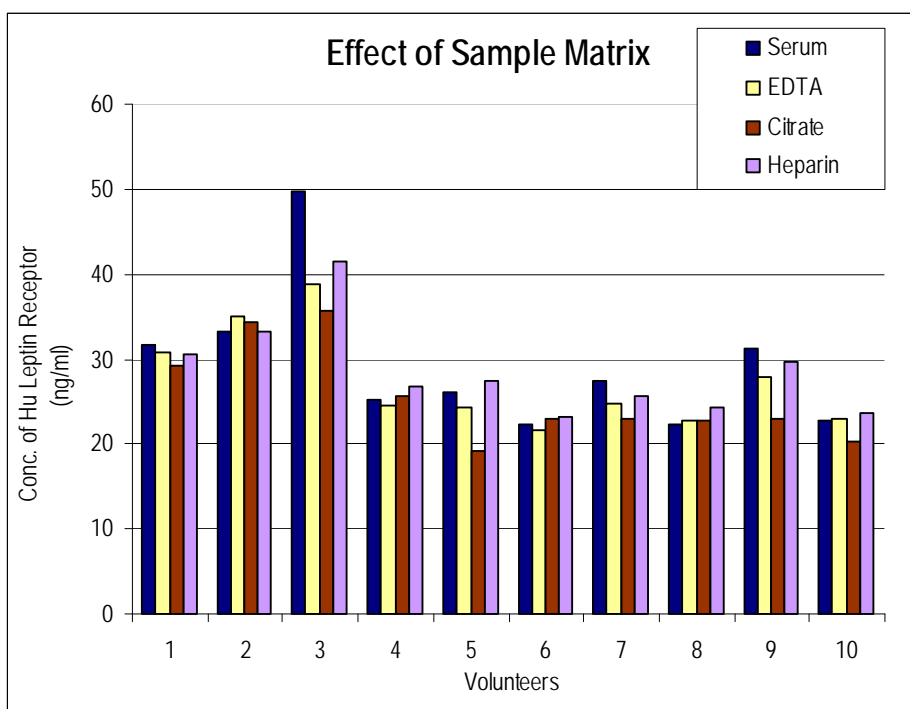


Fig. 3: Leptin receptor levels measured using Human Leptin Receptor ELISA from 10 individuals using serum, EDTA, citrate and heparin plasma, respectively.

- **Stability of samples stored at 2-8°C**

Samples should be stored at -20°C. However, no decline in concentration of leptin receptor was observed in serum and plasma samples after 10 days when stored at 2-8°C. To avoid microbial contamination, samples were treated with ε-aminocaproic acid and sodium azide, resulting in the final concentration of 0.03% and 0.1%, respectively.

Sample	Incubation Temp, Period	Serum (ng/ml)	Plasma (ng/ml)		
			EDTA	Citrate	Heparin
1	-20°C	33.28	35.09	34.46	33.34
	2-8°C, 1 day	35.80	30.82	32.30	33.49
	2-8°C, 10 days	35.15	31.77	35.45	33.96
2	-20°C	49.75	38.82	35.69	41.48
	2-8°C, 1 day	47.13	41.38	41.19	43.00
	2-8°C, 10 days	45.04	46.02	40.49	44.28
3	-20°C	22.36	21.70	22.97	23.27
	2-8°C, 1 day	21.80	21.81	24.11	21.54
	2-8°C, 10 days	23.56	22.93	19.34	22.70

- **Effect of Freezing/Thawing**

No decline was observed in concentration of human leptin receptor in serum and plasma samples after repeated (5x) freeze/thaw cycles. However it is recommended to avoid unnecessary repeated freezing/thawing of the samples.

Sample	Number of f/t cycles	Serum (ng/ml)	Plasma (ng/ml)		
			EDTA	Citrate	Heparin
1	1x	21.39	20.89	16.77	21.78
	3x	18.69	18.31	16.02	22.72
	5x	20.06	20.47	16.43	21.15
2	1x	26.90	24.71	23.33	27.13
	3x	26.77	26.91	22.29	26.58
	5x	25.24	24.80	20.93	24.36
3	1x	18.07	18.51	16.00	18.96
	3x	17.57	17.95	17.20	19.73
	5x	19.39	18.55	17.29	19.92

14. DEFINITION OF THE STANDARD

The Standard used in this kit is recombinant human IgG-Fc-fragment - human OB-R dimeric chimera, which is different from the native soluble OB-R that is measured in human serum. Mature OB-R is a disulfide-linked homodimeric protein. As a result of glycosylation, recombinant human leptin receptor/Fc chimera migrates as a 205 kDa protein in SDS-PAGE and we used to employ the unit U/ml.

From the lot RD-877 we started using the unit ng/ml. 1 ng OB-R/ml = 1 U OB-R/ml (previously used). It can be to recalculate previous results with factor 1.0; e.g. concentration of leptin receptor 24.50 U/ml in a sample measured in previous assays corresponds to 24.50 ng/ml of leptin receptor measured in this assay.

15. METHOD COMPARISON

The BioVendor Human Leptin Receptor ELISA was not compared to the other commercial immunoassays.

16. TROUBLESHOOTING AND FAQS

»» Weak signal in all wells

Possible explanations:

- Omission of a reagent or a step
- Improper preparation or storage of a reagent
- Assay performed before reagents were allowed to come to room temperature
- Improper wavelength when reading absorbance

»» High signal and background in all wells

Possible explanations:

- Improper or inadequate washing
- Overdeveloping; incubation time with Substrate Solution should be decreased before addition of Stop Solution
- Incubation temperature over 30°C

»» High coefficient of variation (CV)

Possible explanation:

- Improper or inadequate washing
- Improper mixing Standards, Quality Controls or samples

17. REFERENCES

»» References to Leptin Receptor:

- Lahlou N. et al.: Soluble leptin receptor in serum of subjects with complete resistance to leptin: *Diabetes*; **19**:1347-1352 (2000)
- Auwerx J. and Staels B.: Leptin (Review article). *The Lancet*; **13**:737 (1998)
- Chen H. et al.: Evidence that the diabetes gene encodes the leptin receptor: identification of a mutation in the leptin receptor gene in db/db mice. *Cell*; **84**:491-495 (1996)
- Ciofii J. A. et al.: Novel B219/OB receptor isoforms: possible role of leptin in hematopoiesis and reproduction. *Nature Med.*; **2**:585-589 (1996)
- Houseknecht KL and Portocarrero CP: Leptin and its receptors: Regulators of whole body energy homeostasis. *Domestic animal endocrinology*; **15 (6)**:457-475 (1998)
- Lee GH et al.: Abnormal splicing of the leptin receptor in diabetic mice. *Nature*; **379**:632-635 (1996)
- Tartaglia LA et al.: Identification and Expression Cloning of a Leptin Receptor, OB-R. *Cell*; **83**:1263-1271 (1995)
- Tartaglia LA: The leptin receptor. *J. Biol. Chem.*; **272**:6093-6096 (1997)

»» References to this product:










- Buyan N, Ozkaya O, Bideci A, Soylemezoglu O, Cinaz P, Gonen S, Kalman S, Bakkaloglu S, Hasanoglu E: Leptin, soluble leptin receptor, and transforming growth factor-beta1 levels in minimal change nephrotic syndrome. *Pediatr Nephrol. Jul*; **29 (8)**: (2003)
- Gavrila A, Peng CK, Chan JL, Mietus JE, Goldberger AL, Mantzoros CS: Diurnal and ultradian dynamics of serum adiponectin in healthy men: comparison with leptin, circulating soluble leptin receptor, and cortisol patterns. *J Clin Endocrinol Metab Jun*; **88(6)**:2838-43 (2003)
- Mann DR, Johnson AO, Gimpel T, Castracane VD: Changes in circulating leptin, leptin receptor, and gonadal hormones from infancy until advanced age in humans. *J Clin Endocrinol Metab Jul*; **88(7)**:3339-45 (2003)
- Sandhofer A, Laimer M, Ebenbichler CF, Kaser S, Paulweber B, Patsch JR: Soluble leptin receptor and soluble receptor-bound fraction of leptin in the metabolic syndrome. *Obes Res Jun*; **11(6)**:760-8 (2003)
- Kado N, Kitawaki J, Koshihara H, Kitaoka Y, Teramoto M, Honjo H: Relationships between the serum levels of soluble leptin receptor and free and bound leptin in non-pregnant women of reproductive age and women undergoing controlled ovarian hyperstimulation. *Hum Reprod Apr*; **18(4)**:715-20 (2003)
- Yannakouli M, Yiannakouris N, Bluher S, Matalas AL, Klimis-Zacas D, Mantzoros CS: Body fat mass and macronutrient intake in relation to circulating soluble leptin receptor, free leptin index, adiponectin, and resistin concentrations in healthy humans. *J Clin Endocrinol Metab Apr*; **88(4)**:1730-6 (2003)

- Rosicka M, Krsek M, Matoulek M, Jarkovska Z, Marek J, Justova V V, Lacinova Z: Serum ghrelin levels in obese patients: the relationship to serum leptin levels and soluble leptin receptors levels. *Physiol Res Feb*; **52**(1):61-66 (2003)
- Pecoits-Filho R, Nordfors L, Heimbürger O, Lindholm B, Anderstam B, Marchlewska A, Stenvinkel P.: Soluble leptin receptors and serum leptin in end-stage renal disease: relationship with inflammation and body composition. *Eur J Clin Invest Nov*; **32**(11):811-7 (2002)
- Chan JL, Blüher S, Yiannakouris N, Suchard MA, Kratzsch J and Mantzoros CS: Regulation of Circulating Soluble Leptin Receptor Levels By Gender, Adiposity, Sex Steroids, and Leptin. *Observational and Interventional Studies in Humans Diabetes*; **51**(7):2105-2112 (2002)
- Laimer M, Ebenbichler CF, Kaser S, Sandhofer A, Weiss H, Nehoda H, Aigner F and Patsch JR: Weight Loss Increases Soluble Leptin Receptor Levels and the Soluble Receptor Bound Fraction of Leptin. *Obesity Res*; **10**(7):597-601 (2002)
- Chatzazoni K., Papathanassopoulos P., Gourzoulidou E. and Mouzaki A.: Leptin and its soluble receptor in plasma of patients suffering from remitting – relapsing multiple sclerosis (MS) In vitro effect of leptin on type-1 and type-2 cytokine secretion by peripheral blood mononuclear cells, T-cells and monocytes Of MS patients. *J. of Autoimmunity*; **23**:169-177 (2004)
- Tsiotra P.C., Pappa V., Koukourava A., Economopoulos T., Tsigos C. and Raptis S.A.: Expression of Leptin Receptors in Mononuclear Cells from Myelodysplastic Syndromes and Acute Myeloid Leucemias. *Acta Haemologica*; **114**:71-77 (2005)
- Szalay F., Folhoffer A., Horváth A., Csak T., Speer G., Nagy Z., Lacatos P., Horváth C., Habior A., Tornai I. and Lakatos P.L.: Serum leptin, soluble leptin receptor, free leptin index and bone mineral density in patients with primary biliary cirrosis. *Eur J. of Gastroenterology & Hepatology*; **17**(9):923-928 (2005)
- Presle, N., Pottier, P., Dumond, H., Guillaume, C., Lapique, F., Pallu, S., Mainard D., Netter, P. & Terlain, B: Differential distribution of adipokines between serum and synovial fluid in patients with osteoarthritis. Contribution of joint tissues to their articular production. *Osteoarthritis and Cartilage*; **14**:690-695 (2006).
- Verhaeghe, J., van Bree, R. & van Herck, E.: Maternal body size and birth weight: can insulin or adipokines do better? *Metabolism*; **55**:339-344 (2006).
- Dostálová, I., Kopský, V., Dušková, J., Papežová, H., Pacák, K. & Nedvídková, J.: Leptin concentrations in the abdominal subcutaneous adipose tissue of patients with anorexia nervosa assessed by in vivo microdialysis. *Regulatory Peptides*; **128**:63-68 (2005).
- Housova J, Anderlova K, Krizova J, Haluzikova D, Kremen J, Kumstyrova T, Papezova H, Haluzik M.: Serum adiponectin and resistin concentrations in patients with restrictive and binge/purge form of anorexia nervosa and bulimia nervosa. *J Clinical Endocrinology & Metabolism*; **90**: 1366-1370 (2005).
- Jarkovská, Z., Hodková, M., Sazamová, M., Rosická, M., Dusilová-Sulková, S., Marek, J., Justová, V., Lacinová, Z., Haluzík, M., Haas, T. & Kršek, M.: Plasma levels of active and total ghrelin in renal failure: A relationship with GH/IGF-I axis. *Growth Hormone & IGF Research*; **15**: 369-376 (2005).


- Papadopoulos, D.P., Makris, T.K., Krespi, P.G., Poulakou, M., Paizis, I.A., Hatzizacharias, A.N., Perrea, D. & Votteas, V.V.: Human soluble leptin receptor number in healthy normotensive individuals with high normal blood pressure. *American Journal of Hypertension*; **18**: 1001-1004 (2005).
- Verhaeghe, J., van Bree, R., Lambin, S., Caluwaerts, S.: Adipokine Profile and C-Reactive Protein in Pregnancy: Effects of Glucose Challenge Response Versus Body Mass Index. *Journal of the Society for Gynecologic Investigation*; **12**: 330-334 (2005).
- Krsek M, Silha JV, Jezkova J, Hana V, Marek J, Weiss V, Stepan JJ, Murphy LJ.: Adipokine levels in Cushing's syndrome; elevated resistin levels in female patients with Cushing's syndrome. *Clinical Endocrinology*; **60**: 350-357 (2004).

➤➤ For more references on this product see our WebPages at www.biovendor.com

18. EXPLANATION OF SYMBOLS

	<p>Catalogue number</p>
	<p>Content</p>
	<p>Lot number</p>
	<p>See instructions for use</p>
	<p>Biological hazard</p>
	<p>Expiry date</p>
	<p>Storage conditions</p>
	<p>Identification of packaging materials</p>
	<p>In vitro diagnostic medical device</p>


Assay Procedure Summary

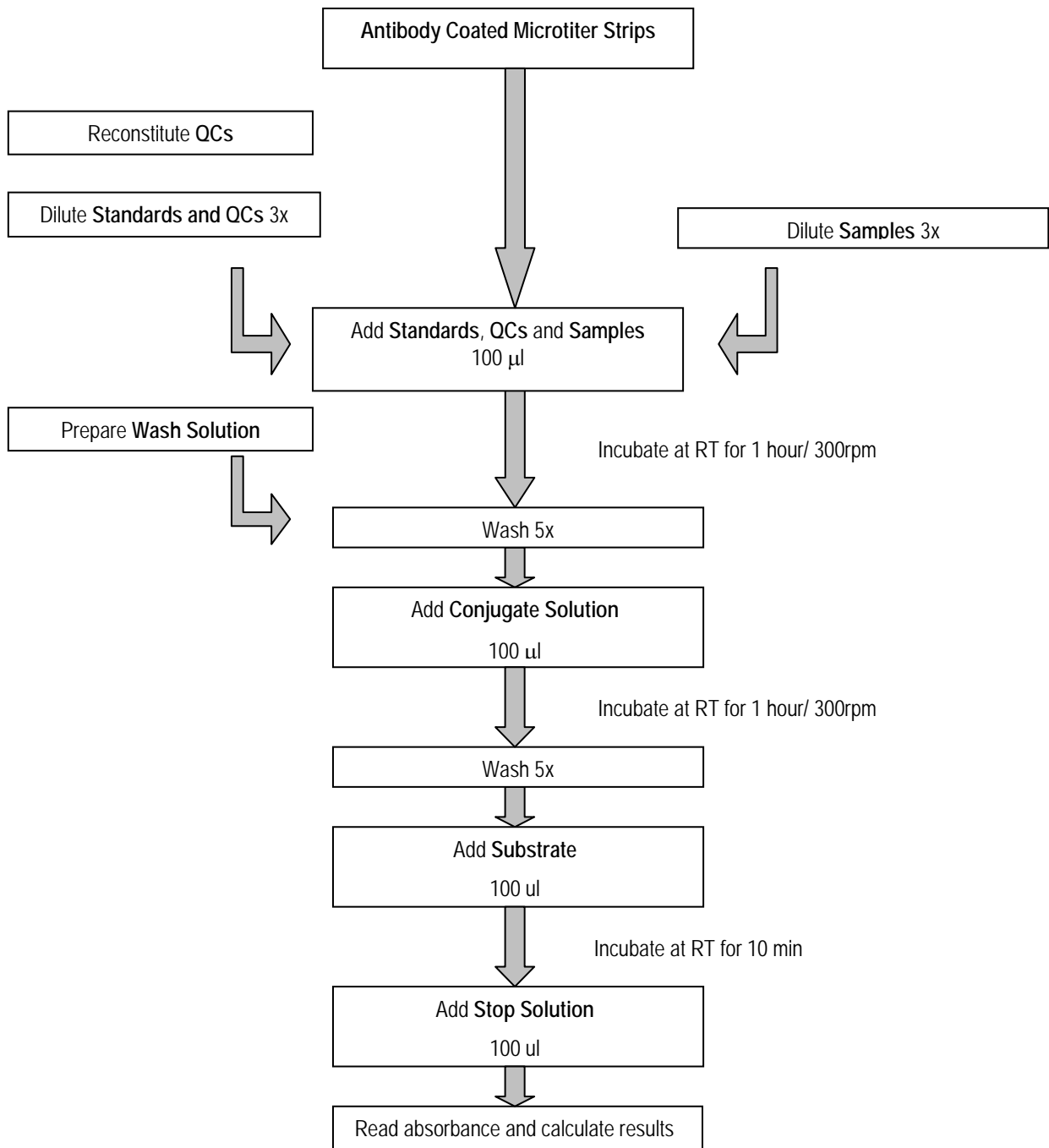

12								
11								
10								
9								
8								
7								
6								
5								
4								
3								
2								
1								
	A	B	C	D	E	F	G	H

NOTES



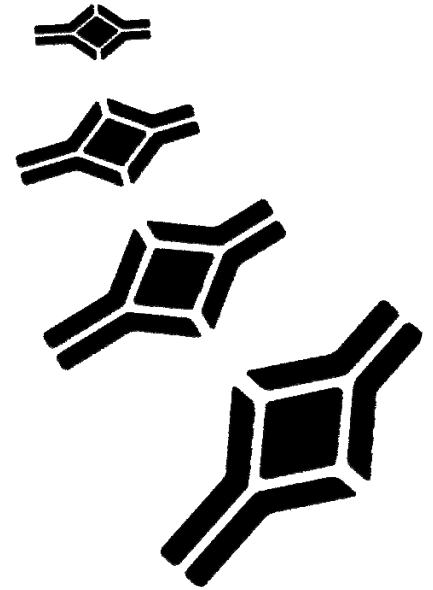

HEADQUARTERS:
BioVendor Laboratorní
medicína, a.s.

CTPark Modrice
Evropska 873

664 42 Modrice
CZECH REPUBLIC

Phone: +420-549-124-185
Fax: +420-549-211-460

E-mail: info@biovendor.com
Web: www.biovendor.com

EUROPEAN UNION:
BioVendor GmbH

Im Neuenheimer Feld 583

D-69120 Heidelberg
GERMANY

Phone: +49-6221-433-9100
Fax: +49-6221-433-9111

E-mail: infEU@biovendor.com

USA, CANADA AND MEXICO:
BioVendor LLC

1463 Sand Hill Road
Suite 227

Candler, NC 28715
USA

Phone: +1-828-670-7807
+1-800-404-7807
Fax: +1-828-670-7809

E-mail: infoUSA@biovendor.com

CHINA - Hong Kong Office:
BioVendor Laboratories Ltd

Room 4008
Hong Kong Plaza, No.188

Connaught Road West
Hong Kong, CHINA

Phone: +852-2803-0523
Fax: +852-2803-0525

E-mail: infoHK@ovendor.com

CHINA – Mainland Office:
BioVendor Laboratories Ltd

Room 2405 YiYa Tower
TianYu Garden, No.150

Lihe Zhong Road
Guang Zhou, CHINA

Phone: +86-20-3884-0399
+86-20-3884-0386
Fax: +86-20-3884-0386

E-mail: infoCN@ovendor.com